


capillus

NEURO

**Neuro Red & Near-Infrared Technology
for Cognitive Wellness**

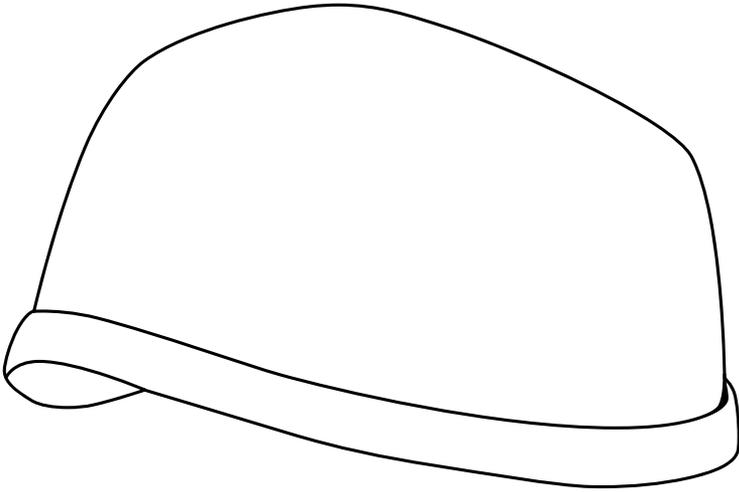


User Manual

Table of Contents

What is in the box	1
About the Capillus®NEURO	2
Indications of Use	2
Frequency of Use	2
Duration of Use	2
Warnings & Precaution	2
Safety Labels	4
Instructions for Use	4
Cleaning Instructions	5
Servicing	6
Storage Instructions	6
Disposal Instructions	7
Return Policy	7
Manufacturer's Guarantee	7
Risk of Loss, Other Terms of Sale	7
How to Contact Us	8
Device Specifications	9

What is in the box

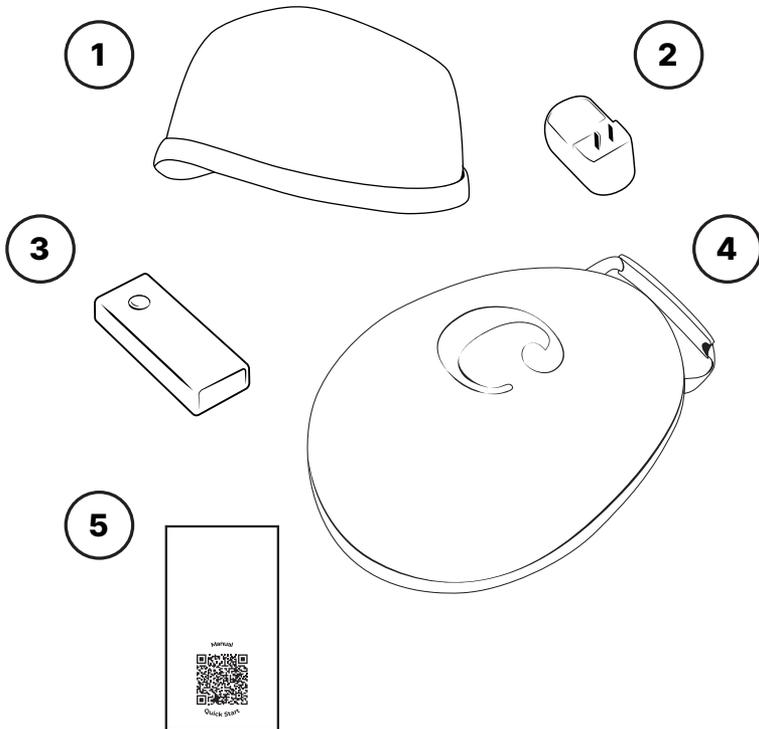

1. Capillus® Healing Pod
2. AC adapter
3. Rechargeable Battery

4. Carrying Case
5. QR Code with User Guide and Instruction Manual

About the Capillus® NEURO

The Capillus Neuro is engineered with an anatomically distributed array of medical-grade near-infrared laser diodes positioned to correspond with major functional regions of the cerebral cortex. This regional design enables broad and consistent light delivery across cortical areas associated with cognition, memory, attention, emotional regulation, and sensory integration.

Indications of Use

The Capillus® NEURO is an over the counter, at-home device. Providing 12 minutes/ session, the Capillus NEURO delivers a gentle experience that promote general cognitive wellness. The Capillus Neuro is intended to support general cognitive wellness and mental performance in healthy individuals through non-invasive photobiomodulation. It is not intended for use in the diagnosis, mitigation, treatment, or prevention of any disease.

Frequency of Use

For optimal results, it is recommended to use the Capillus® NEURO 12 minute pre-programed sessions for twice a day. Exceeding the recommended frequency may not increase the effect of the device. .

Duration of Use

The Capillus® NEURO may be used indefinitely, provided it is operated according to its instructions.

Warnings & Precautions

Do not use the Capillus® NEURO in:

- Conjunction with medications that are photo sensitizing.
- Products that are photosensitizing.

Immediately stop using the device if you experience any discomfort.

Precautions - Consult a medical professional before using the Capillus® NEURO if::

- You are experiencing skin sensitivity or irritation.
- CAUTION: The light emitted from the Capillus® NEURO may be harmful to the eyes. Do not stare at the light source.
- Do not use the Capillus® NEURO if it has been left outdoors.

To reduce the risk of fire, thermal and chemical burns, electric shock, serious injury, follow the precautions in this section and the procedures in this manual.

Charge Safely

- Before initial use, please charge the Battery Pack overnight as per the following instructions for optimal battery life. Only charge the Battery Pack with the provided AC adapter.
- Follow the manufacturer's "Instructions for use" to charge the battery of your Capillus® NEURO.
- Check if the voltage indicated on the equipment corresponds to the local mains voltage before you connect the equipment to avoid risk of electrocution or permanent damage to the equipment.
- Insert provided AC adapter into power outlet and then connect Battery Pack to AC adapter. Make sure access to AC adapter is unobstructed, for safety. The indicator lights on the top of the Battery Pack will indicate charge level.
- Once all four indicator lights are turned on and static (no blinking), the Battery Pack is ready for use. Disconnect Battery Pack from AC adapter for use. Disconnect AC adapter from power outlet. We recommend storing all parts of your device in the Carrying Case.
- Do not charge the Capillus® NEURO in the bathroom or near water.
- Do not charge the Capillus® NEURO after contact with water.
- Store the battery and the charging cord in a safe and dry place.

Supervision and Intended users:

- Do not use if any of the conditions in "Warnings & Precautions" section apply to you.
- Do not leave the Capillus® NEURO unattended when it is powered on.
- The Capillus® NEURO is not intended for use by people with reduced physical, sensory, or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instructions concerning use of the equipment. Please refer to the "Warnings & Precautions" before use.
- The Capillus® NEURO is not intended for use by children under 18 years old. Children should be supervised to ensure they do not play with the Capillus® NEURO.
- Small parts can be inhaled or swallowed by children—if any small parts break off or detach from the product, dispose of it safely.
- The Capillus® NEURO is intended to be used by only one person.

Safety Labels

	Indicates Manufacturing Company.
	Indicates the device should not be disposed of in the normal waste stream; it should be taken to a recycling center for proper disposal.
	Indicates Laser Light Radiation as described on label.
	Indicates Type BF applied part (60335); B=Body, F=Floating applied part. No conductive contact with USER.
	Indicates a medical device that needs to be protected from moisture.
IP22	Indicates minimum protection against dust and moisture. Please use the carrying case for protection between uses.

The Capillus® NEURO gives off no heat or harmful radiation, causes no pain, and does not require the use of drugs or topical applications.

EYE SAFETY CAUTION: Class 3R 5 MilliWatts, 650 Nanometers Red and 808 Near Infrared Laser Diodes. DO NOT STARE DIRECTLY INTO THE LASER LIGHT FOR EXTENDED PERIODS.

Instructions for Use

Please read the Warnings & Precautions and Instructions before first use.

Check the Product for Damage before each use.

- Inspect the Capillus® NEURO for cleanliness, cracks, or sharp edges.
- Check the Capillus® NEURO for signs of corrosion.
- These conditions can irritate or damage your skin.

1. Unpack the components and make sure all the parts are present according to “What is in the box” section. If there are parts missing, contact customer service at 1-888-470-2710 or support@capillus.com.
2. To power on, press the power button for 1 second located in front of the battery and place the Neuro device on your head.
3. The emission indicator light on top of the battery will glow to indicate that treatment has begun and lasers are operational. Please note that there is a sensor on the inside of the device that only allows light to be emitted when the cap is on the head. Should you hear continuous beeping during treatment it may be an indication that treatment has been disrupted. If treatment is interrupted for 10 minutes or longer, the treatment session will be reinitiated upon start. NOTE: The Capillus® laser device will not operate if the battery is connected to the AC adapter.

4. When the automatically programmed treatment session is finished (12 minutes), you will hear three longer successive beeps to indicate the treatment period is over, the lasers will automatically shut off, and the emission indicator light on the battery will turn off.
5. When treatment is over after 12 minutes, the battery will turn off automatically. Disconnect Battery Pack from Neuro device and store until next use.
6. You may repeat the same procedure twice a day. Allow the device to fully cool down before each use to ensure device optimal performance.

Charging Instructions

1. Make sure your Capillus® NEURO is fully charged. Please refer to the 'Charging Instructions' on how to charge your device.
2. For optimal battery life, Battery Pack should be charged overnight before first use until all 4 indicator lights on top of Battery Pack are on and static.
3. The battery should also be disconnected from laser device between uses for optimal battery life and performance. After initial charge, please charge battery, only when one indicator light is left.
4. Connect the laser device to the battery by matching up the small white indicator arrows on the cables to each component. **Important: The arrows must be aligned for the cables to make a proper connection. Once the arrows are aligned, push cables together and secure.**

Cleaning Instructions

If it is necessary to clean the surfaces of your Capillus® NEURO, please follow these instructions. Note: The Capillus® NEURO has no parts that can be serviced by the user. If service is required, contact Capillus® at 1-888-571-5334.

MAKE SURE THE DEVICE IS TURNED OFF UNPLUG THE CAPILLUS® NEURO FROM ANY POWER CORD BEFORE CLEANING.

- To clean the Capillus® NEURO, only clean the silicone and outer portion of the Capillus® NEURO.
- Gently wipe the device with a damp cloth clean the device. Allow it to air dry or wipe down with a dry cloth and make sure to keep it out of direct sunlight.
- Do not use excessive pressure or force as this may damage the unit. Do not use acetone or any other solvents on any part of the device. Acetone and other solvents can damage the unit and void the warranty.

Keep It Dry.

- Keep the Capillus® NEURO dry. If it gets wet, do not charge or use- the battery may be damaged. Follow disposal instructions.
- Do not use the NEURO in wet surroundings (e.g., in a bathtub, shower, or swimming pool).
- Do not submerge the Capillus® NEURO in water.
- Do not let water run into the Capillus® NEURO.
- It is very important to keep the Capillus® NEURO dry. Any contact with water could damage the device.
- Do not use or store the Capillus® NEURO in the bathroom or near water.
- Do not charge or use the Capillus® NEURO if it has been left outdoors.

Servicing

Protect the Battery from Damage.

- Do not try to open the Capillus® NEURO or the battery.
- Do not crush, drop, or damage the Capillus® NEURO.
- Stop using or disconnect any power cable if the Capillus® NEURO feels unusually hot or emits smoke or a bad smell.
- Follow “Disposal Instructions”.

Do Not Repair or Modify.

- Do not attempt to repair or modify the Capillus® NEURO.
- Do not crush or dispose of in fire.

Storage Instructions

Transport and Storage Conditions.

- Only transport and store the Capillus® NEURO in the following environmental conditions: Keep between 0°C (32°F) and 25°C (77°F).
- Do not use the Capillus® NEURO outdoors. Only use the Capillus® NEURO indoors under the following environmental conditions Temperature: between 10°C and 30°C.
- A relative humidity range of 15% to 90% atmospheric pressure range of 700 Pa to 1060Pa.

Disposal Instructions

Dispose of it in appropriate waste stream as per local ordinance and regulations.

Return Policy

All www.Capillus.com sales are final. No refunds or exchanges with the exception of the Manufacturer's Guarantee. If you wish to make a return as per above, contact Capillus® at 1 (786) 888-6249 to obtain a Return Material Authorization (RMA). Capillus® will not accept any shipments without an RMA. We recommend that you insure your shipment and register to receive tracking updates. Capillus® will only accept fully functional, like-new devices for returns. Capillus® will provide credit to your credit card or eligible refund via check.

If you received any products in a damaged condition, hang on to your shipment and DO NOT discard any damaged boxes, packaging, or products. Report the damage to us and we will determine if you should return the product for a replacement. You may be asked to provide pictures of your damage to help with the processing of your claim.

International orders will not receive any refunds for the shipping costs paid in the initial order. The refund will only be applicable to the price of the products.

Manufacturer's Guarantee

Capillus® warrants the device against defects in material or workmanship for a period of 3 years. For more info visit: <https://www.Capillus.com/pages/terms-conditions>. If you already have a Capillus® NEURO. Do not forget to Register your Warranty at www.Capillus.com/warranty

If a defect in material or workmanship exists, Capillus® will repair or replace the necessary parts with new or reworked parts at no charge. Any replacement part will be covered under the original limited warranty. This Warranty is not transferable and does not cover damages caused by acts of God, misuse, negligence, accident, or modification of any part of the unit as determined by our service department. This Warranty does not cover damage due to improper operation or maintenance, connection to improper voltage supply, or attempted repair by anyone other than the company. Failure to use and maintain the device in accordance with the instructions in this manual will void the product warranty.

Please contact Capillus® customer service should you need to get warranty service or order replacement parts such as power adapter.

Do not disassemble any part of the Capillus® NEURO and don't attempt to repair the device by yourself. Disassembling the device may damage it or may cause injury to you. If the Capillus® NEURO is damaged, malfunctions, or comes in contact with liquid, contact Capillus®.

Risk of Loss, Other Terms of Sale

The risk of loss and title for items purchased by you pass to you upon our delivery of the items to the carrier. We do not take title to returned items until the item arrives at our fulfillment center. At our discretion, a refund may be issued without requiring a return. In this situation, we do not take title to the refunded item.

NO RESALE. Buyer shall use the device delivered to buyer for the buyer's user only, and shall not resell, transfer, exchange, or otherwise assign the device without first obtaining the prior written consent of Capillus®, which consent may be granted or withheld by Capillus® in its sole discretion.

How to Contact Us

**Address: Curallux, LLC
1715 NW 82 Ave,
Miami, FL 33126**

Customer Service:

**Email: info@Capillus.com
Phone: 1-888-571-5334
web: capillus.com**

Device Specifications

Product Name:	Capillus® NEURO
Model:	NEURO2026A
Total Output:	700mW
Light Source Wavelength:	Red 660nm, Near-Infrared 850nm
Timer:	12-mins sessions
Number of diodes:	140 Laser diodes
AC Adapter:	AC100-240V, 50/60Hz, 0.6A

capillus

RED LIGHT THERAPY